



# DermaBind<sup>SL</sup>

Sterilized, Dehydrated Amniotic Allograft



## DESCRIPTION

DermaBind SL is a dehydrated amniotic membrane covering that preserves the comprehensive collagen matrix, glycoconjugates, and glycosaminoglycans.

## THE DERMABIND PROCESS

DermaBind SL undergoes a proprietary preservation method, which retains all native layers of the placental membrane, including the amnion and chorion with the spongy layer intact, unlike many placental allografts on the market. DermaBind SL undergoes a proprietary preservation method retaining the native extracellular matrix structure of the amniotic membrane, unlike many amniotic allografts on the market.

The DermaBind process does not use unnecessary chemicals, antibiotics or preservatives and is never frozen.

The finished product is physician-friendly, durable, easily applied to a wound and can be stored at room temperature for up to 3 years.



### AMNIOTIC MEMBRANE

- Collagens I, III, IV, V, VI
- Proteoglycans
- Glycoproteins
- Glycosaminoglycans
- Fibronectin
- Nidogen
- Laminin
- Semi-permeable biological barrier

## COMPOSITION

DermaBind SL contains 400+ proteins, including collagens, fibrins, elastins, glucosaminoglycans and glycoconjugates.

HealthTech Wound Care, Inc.  
healthtechwoundcare.com  
833-381-2402

sales@worldreachhealth.com

© 2023

Exclusively distributed by  
World Reach Health, LLC

3501 W. Algonquin Rd | Suite 135  
Rolling Meadows, IL 60008  
WorldReachHealth.com



847-220-4664

Manufactured By



## INTENDED USE

DermaBind SL can be used as a protective wound covering as either a partial and full thickness acute and chronic wounds.

- Trauma Wounds
- Dehisced Wounds
- Venous Leg Ulcers (VLUs)
- Pressure Injuries
- Wounds with Exposed Bone/Tendon
- Diabetic Foot Ulcers (DFUs)


Can be used at the onset and for the duration of the wound, weekly or at the discretion of the treating physician.

## APPLICATION

- Using aseptic technique, remove DermaBind SL from the package; the product may be trimmed as necessary
- Place the DermaBind SL patch onto the wound
- Assure DermaBind SL is in intimate contact with the wound bed; the product can be hydrated once placed on the wound bed
- Adhere DermaBind SL using preferred fixation method, then cover DermaBind SL with a primary non-adherent dressing
- Apply secondary dressings specific to the wound type over the non-adherent layer, which should remain in place for up to one week

## SIZE & STORAGE

- Available in a wide range of shapes and sizes
- Maintain at ambient temperature prior to patient application
- Do not freeze
- May be stored for up to 3 years at room temperature.

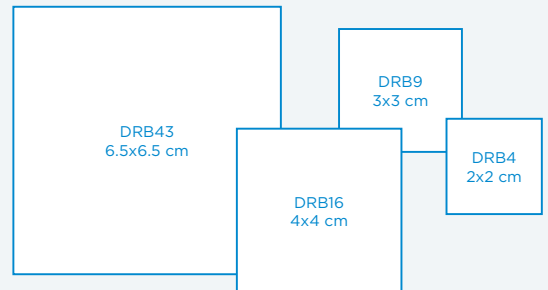

## REIMBURSEMENT

- Medicare contractors may cover DermaBind SL for VLUs and DFUs<sup>†</sup>
- DermaBind SL can be applied to, and billed for, other wound types
- Confirm coverage prior to treatment through our Benefits Program

<sup>†</sup>Consult with your LCD (local coverage determination) for the specific wound types covered. VLU = venous leg ulcer; DFU = diabetic foot ulcer

**DermaBind SL  
billing**

**Q-4284**

### CPT codes

Site Preparation CPT<sup>†</sup>  
15002-15005

Application CPT codes<sup>†</sup>  
15271-15278

<sup>†</sup>Subject to guidelines and approvals, please contact our billing team for further details

**BILLING SUPPORT**  
**REIMBURSEMENT@HEALTHTECHWC.COM**

HealthTech Wound Care, Inc.  
healthtechwoundcare.com  
833-381-2402

sales@worldreachhealth.com

© 2023

Exclusively distributed by  
World Reach Health, LLC

3501 W. Algonquin Rd | Suite 135  
Rolling Meadows, IL 60008  
WorldReachHealth.com



847-220-4664

Manufactured By

