



DESCRIPTION

DermaBind FM is a dehydrated amniotic membrane covering that preserves the comprehensive collagen matrix, glycoconjugates, and glycosaminoglycans.

THE DERMABIND FM PROCESS

DermaBind FM undergoes a proprietary preservation method, which retains all native layers of the placental membrane, including the amnion and chorion with the spongy layer intact, unlike many placental allografts on the market.

The DermaBind FM process does not use unnecessary chemicals, antibiotics or preservatives and is never frozen.

The finished product is physician-friendly, durable, easily applied to a wound and can be stored at room temperature for up to 3 years.



COMPOSITION

DermaBind FM contains 400+ proteins, including collagens, fibrins, elastins, glucosaminoglycans and glycoconjugates.

INTENDED USE

DermaBind FM can be used as a protective wound covering as either a partial and full thickness acute and chronic wounds.

- Trauma Wounds
- Dehisced Wounds
- Diabetic Foot Ulcers (DFUs)
- Pressure Injuries
- Venous Leg Ulcers (VLUs)


Can be used at the onset and for the duration of the wound, weekly or at the discretion of the treating physician.

APPLICATION

- Using aseptic technique, remove DermaBind FM from the package; the product may be trimmed as necessary
- Place the DermaBind FM patch onto the wound
- Assure DermaBind FM is in intimate contact with the wound bed; the product can be hydrated once placed on the wound bed
- Adhere DermaBind FM using preferred fixation method, then cover DermaBind FM with a primary non-adherent dressing
- Apply secondary dressings specific to the wound type over the non-adherent layer, which should remain in place for up to one week

SIZE & STORAGE

- Available in a wide range of shapes and sizes
- Maintain at ambient temperature prior to patient application
- Do not freeze
- May be stored for up to 3 years at room temperature.

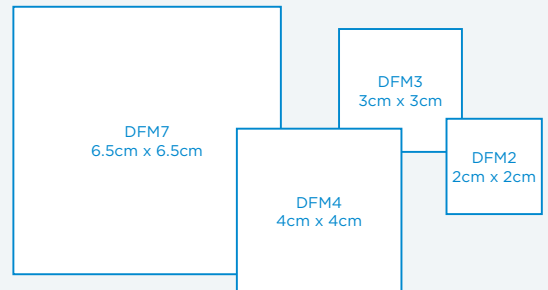

REIMBURSEMENT

- Medicare contractors cover DermaBind FM for VLUs and DFUs†
- DermaBind FM can be applied to, and billed for, other wound types
- Confirm coverage prior to treatment through our Benefits Program

†Consult with your LCD (local coverage determination) for the specific wound types covered. VLU = venous leg ulcer; DFU = diabetic foot ulcer

**DermaBind FM
billing**

Q 4313

CPT codes

Site Preparation CPT[‡]
15002-15005

Application CPT codes[‡]
15271-15278

[‡]Subject to guidelines and approvals, please contact our billing team for further details

BILLING SUPPORT
REIMBURSEMENT@HEALTHTECHWC.COM

Manufactured By



HealthTech Wound Care, Inc.

© 2023

healthtechwoundcare.com
833-381-2402

Exclusively distributed by
World Reach Health, LLC
sales@worldreachhealth.com