



DermaBind TL

Sterilized, Dehydrated Amniotic Allograft



DESCRIPTION

DermaBind TL is a dehydrated amniotic membrane covering that preserves the comprehensive collagen matrix, glycoconjugates, and glycosaminoglycans.

THE DERMABIND TL PROCESS

DermaBind TL undergoes a proprietary preservation method, which retains all native layers of the placental membrane, including the amnion and chorion with the spongy layer intact, unlike many placental allografts on the market.

The DermaBind TL process does not use unnecessary chemicals, antibiotics or preservatives and is never frozen.

The finished product is physician-friendly, durable, easily applied to a wound and can be stored at room temperature for up to 3 years.



COMPOSITION

DermaBind TL contains 400+ proteins, including collagens, fibrins, elastins, glucosaminoglycans and glycoconjugates.

Manufactured By



HealthTech Wound Care, Inc.
healthtechwoundcare.com
833-381-2402

© 2023

Exclusively distributed by



World Reach Health, LLC
WorldReachHealth.com
847-220-4664

sales@worldreachhealth.com
3501 W. Algonquin Rd | Suite 135
Rolling Meadows, IL 60008

INTENDED USE

DermaBind TL can be used as a protective wound covering as either a partial and full thickness acute and chronic wounds.

- Trauma Wounds
- Dehisced Wounds
- Venous Leg Ulcers (VLUs)
- Pressure Injuries
- Wounds with Exposed Bone/Tendon
- Diabetic Foot Ulcers (DFUs)


Can be used at the onset and for the duration of the wound, weekly or at the discretion of the treating physician.

APPLICATION

- Using aseptic technique, remove DermaBind TL from the package; the product may be trimmed as necessary
- Place the DermaBind TL patch onto the wound
- Assure DermaBind TL is in intimate contact with the wound bed; the product can be hydrated once placed on the wound bed
- Adhere DermaBind TL using preferred fixation method, then cover DermaBind TL with a primary non-adherent dressing
- Apply secondary dressings specific to the wound type over the non-adherent layer, which should remain in place for up to one week

SIZE & STORAGE

- Available in a wide range of shapes and sizes
- Maintain at ambient temperature prior to patient application
- Do not freeze
- May be stored for up to 3 years at room temperature.

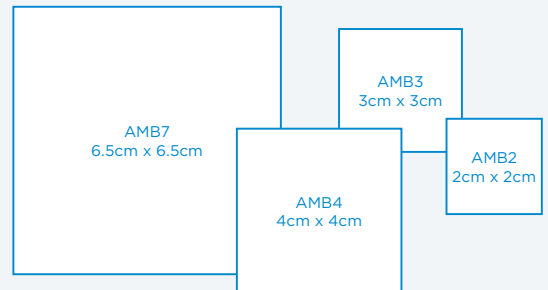

REIMBURSEMENT

- Medicare contractors cover DermaBind TL for VLUs and DFUs*
- DermaBind TL can be applied to, and billed for, other wound types
- Confirm coverage prior to treatment through our Benefits Program

*Consult with your LCD (local coverage determination) for the specific wound types covered. VLU = venous leg ulcer; DFU = diabetic foot ulcer

**DermaBind TL
billing**

Q 4225

CPT codes

Site Preparation CPT[†]
15002-15005

Application CPT codes[†]
15271-15278

[†]Subject to guidelines and approvals, please contact our billing team for further details

BILLING SUPPORT
REIMBURSEMENT@HEALTHTECHWC.COM

Manufactured By



HealthTech Wound Care, Inc.
healthtechwoundcare.com
833-381-2402
© 2023

Exclusively distributed by



World Reach Health, LLC
WorldReachHealth.com
847-220-4664

sales@worldreachhealth.com
3501 W. Algonquin Rd | Suite 135
Rolling Meadows, IL 60008